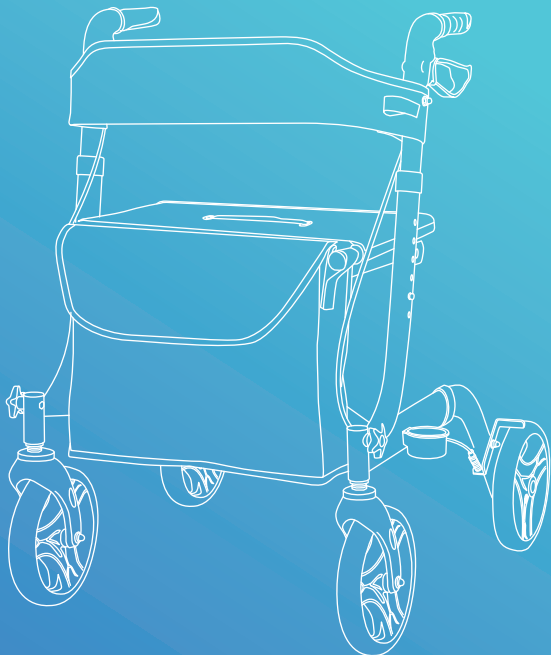
**SAFE AGE**

**PREMIUM ROLLATOR**

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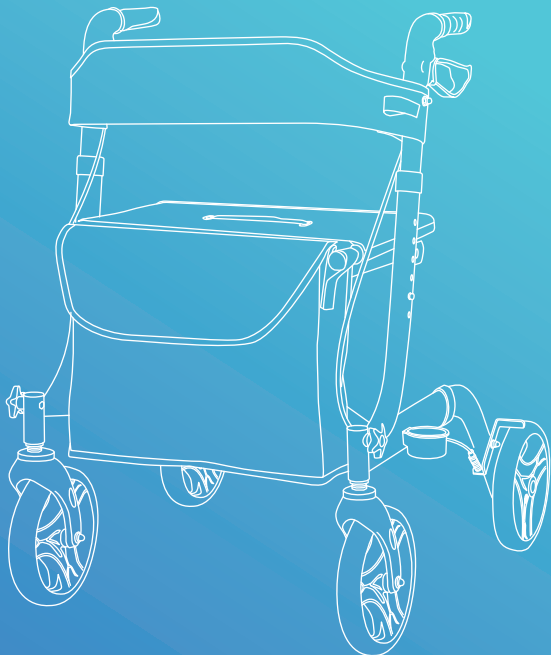
**USER MANUAL**

## SPECIFICATION

<b>Width inside handgrips</b>	<b>  50 cm</b>
<b>Overall width</b>	<b>  61,5 cm</b>
<b>Overall depth</b>	<b>  70 cm</b>
<b>Seat Area</b>	<b>  46 x 25 cm</b>
<b>Seat Height</b>	<b>  55 cm</b>
<b>Adjustable handle height</b>	<b>  82-95 cm</b>
<b>Max. user weight</b>	<b>  136 kg</b>
<b>Weight</b>	<b>  6,8 kg</b>

## ASSEMBLY INSTRUCTION

1. Remove all the packaging. The rollator will be folded as figure 1 shown. Firstly, you should assemble the front wheel with the screw, as figure 2 shown.

2. Taking up the rear tube as shown below and secure the legs with the spring bottom. Always take care to ensure the spring bottom locks in the right position.

## HANDLE ADJUSTABLE

The rollator's handles can be adjusted in height to suit different users. Simply depress the spring button and set to the desired height. Repeat for the opposite side and make sure that both handles are secure and set at the same height before use.

## BRAKE OPERATION & ADJUSTMENT

1. Pull up on brake handle lever to slow the rollator 2. Push down on the brake handle lever to lock the rear wheels, pull up on the brake lever handle to release the lock 3. Brakes must be applied in the lock position when the seat is going to be used, pre check the lock by pushing forward on the rollator.

**Brake adjustment is carried out as follows:**

Turn the adjust screw anti-clockwise to increase the tension then lock the adjustment locking nut clockwise whilst holding the adjuster screw stationary. Brake adjustment reduced to allow light pressure may be ineffective on the brake position.

## MAINTENANCE

Over a period of time the brake cables may stretch, and if no further adjustments can be made the cables should be replaced. Seek the help of your dealer when replacing brake cables. The castors should be checked that they are secure each time before using the rollator. Also, check the brakes operate correctly and the handles and hand knobs are secure. Should any part of the rollator appear loose or insecure discontinue use until it is inspected by an authorised dealer. The following shows maintenance schedule:

### Weekly

- Check brake for malfunction
- Check all wheels and castors for loose bolts and nuts and knobs

### Periodically

- Check handgrips for wear
- Check frame bolts are secure
- Check front and rear axle bolts are secure
- Check tyres for wear or cracks

### Yearly

- Once a year take your rollator to an authorised repairer for a full check, clean, service and lubrication of all moving parts

### Serviceable Components

- Räder
- Wheels
- Hand grips
- Brake Cables
- Brake Lever
- Bearings
- Knobs

## CLEANING

Your rollator can be cleaned with a damp cloth or sponge and warm soapy water. Domestic anti-bacterial surface cleaners may be used if required. Care should be taken not to leave any cleaning residues on the handgrips or seat (where fitted) which may leave them slippery when used. Do not use scouring pads or scrapers as this may damage surface coatings.

## WARNINGS!

1. Maximum working limit 130 kg 2. Do NOT stand on this product. 3. Ensure both push handles are adjusted to the same height 4. Seek advice from a trained health care professional or approved Freedom Healthcare dealer for height adjustment of this product. 5. Always maintain a safe distance between your rollator and other pedestrians or objects. 6. Some components may cause harm or damage when they come into contact with other objects. 7. Do not attempt to replace the wheels or castors of your rollator. If they are faulty or require replacement please contact an authorised repairer. 8. Ensure that your rollator is opened fully (folding scissor arm is straight) before using it.

## INTRODUCTION

The rollators are manufactured from high quality materials and includes seat, bag and locking brakes. The handles are adjustable in height and the rollator folds easily for transport or storage.

## INTENDED USE

Designed to assist the user to walk, rest and carry small items of shopping for both indoor and outdoor use on flat surfaces by a single individual with limited mobility with a maximum weight as shown below. The intended user should possess the necessary physical, visual and cognitive skills to operate the rollator safely. If you have any doubt, please consult your dealer/prescriber. It is important that the user reads and understands the safety advice given within this manual before attempting to operate the product to avoid possible injury to the user or bystanders and avoid damage to the rollator or surrounding property.

## SAFTY ADVICE

This rollator is designed to be used as a mobility aid, creating a natural gate as the user walks securely with their primary weight between the handlebars with the unit in front of the user. Read this manual carefully before using your rollator.

### DO

(1) Always ensure the brakes are in the locked position if the rollator is left unattended (2) Check all castors and wheels are secure before use (3) Check the brakes are functioning properly (4) Check handle height adjustment hand knobs are secure before use (5) Check the rollator is fully unfolded (6) Take care when carrying the rollator when folded. (7) Keep your hands away from the folding mechanism to prevent any risk of pinching.

## DO NOT

(1) Use the rollator as an aid to transport persons, it is not designed as a complete wheelchair (2) Sit on the rollator if the brakes are not locked, this could cause the rollator to move resulting in injury to the user or other persons (3) 'Drop' into the seat of the rollator, always sit into the rollator in a smooth, controlled motion. Seek assistance if necessary. (4) Hang objects on the brake levers or handgrips. This could affect brake operation and/or stability of the rollator.

Every care has been taken to ensure that your rollator will give you safe, reliable service provided you adhere to the advice given above.

## LIST OF COMPONENTS

1. Backrest
2. Shopping bag
3. quick-release locking screw
4. Front fork
5. 8" wheel
6. Pushhandles
7. Adjustable parking/running brake
8. Seat area
9. Spring bottom for handle height adjustment
10. Spring bottom for taking out the rear tube

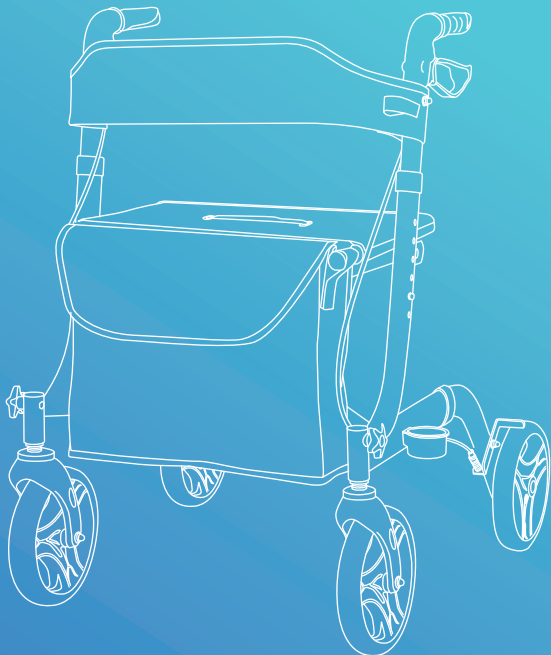
Your rollator is equipped with various components, You should know these components and parts before proceeding to read this user manual. Designed and specifications could be changed without further notices

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