VULPES GOODS

BABYCARE

ELECTRIC BREAST PUMP

PRO

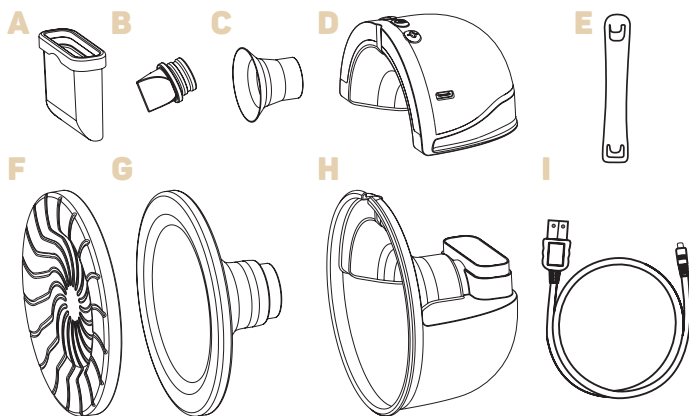
**MANUAL • HANDLEIDING
ANLEITUG • MANUEL**

WWW.VULPESGOODS.COM

ENGLISH	3
Product Information	4
Warnings	13
FAQ	16
Cleaning	17
Product parameters	18
NEDERLANDS	19
Product informatie	20
Waarschuwingen	29
FAQ	32
Schoonmaak	33
Product parameters	34
DEUTSCH	35
Produktinformation	36
Warnungen	45
FAQ	48
Reinigung	49
Produktparameter	50
FRANÇAIS	51
Produit Informations	52
Avertissements	61
FAQ	64
Nettoyage	65
Paramètres du produit	66

ENGLISH

PRODUCT INFORMATION


	NAME OF PARTS	MATERIAL	HEAT-RESISTANCE TEMPERATURE
A	Gasstorage	Silicone	-20~180 °C
B	Duckbill valve	Silicone	-20~180 °C
C	Breast peelers	Silicone	-20~180 °C
D	Engine / Pump unit	ABS/Silicone	/
E	Bra adjusment buckel	/	/
F	Shut-off valve	ABS	/
G	Borstschild	Silicone	-20~180 °C
H	Breast shield	Silicone	-20~180 °C
I	Charging cable	/	/

Assembly of the breast pump

When you receive the pump, the 24mm breast shield reducer is already inserted in the 28mm breast shield. Should you wish to clean the pump extra before use, it is useful to know how to assemble it correctly next. Assembling the breast pump and all its parts correctly is crucial for optimal use. Follow the steps carefully and make sure each part is correctly positioned.


Step 1: Installation of the gas storage

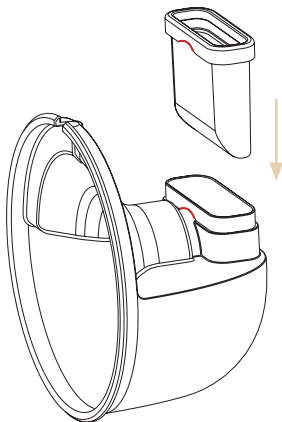The gas storage is placed at the back of the collection tray. Make sure the bottom is correctly fitted with the **rounding** towards the collection tray.


Step 2: Positioning the valve

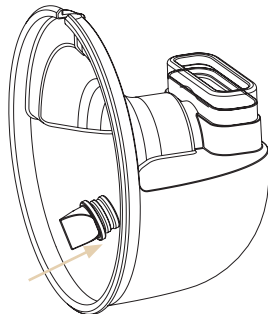The valve is placed in the receptacle of the flask. Make sure the valve is fully pressed on. This will prevent the milk from flowing back.


Step 3: Placement of the breast shield

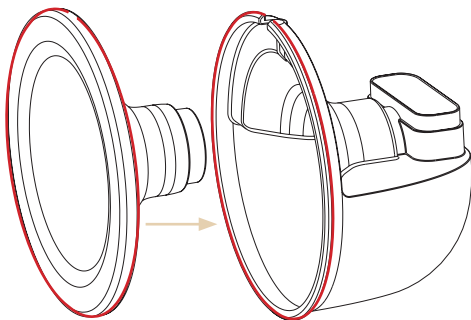The breast shield is placed on the collection tray. It is crucial that the breast shield is fully pressed on and fits over the **edge** of the receptacle.


Step 4: Installing the motor

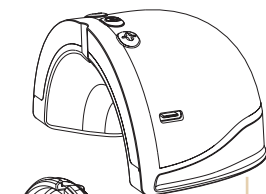
The motor is attached to the back of the flask. Press the motor firmly onto the gas storage.

Step 5: Inserting the breast shield reducers

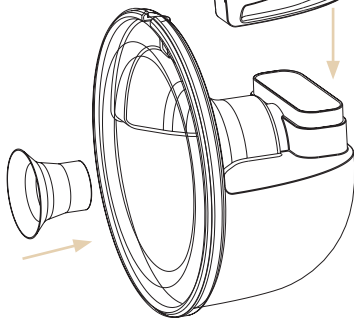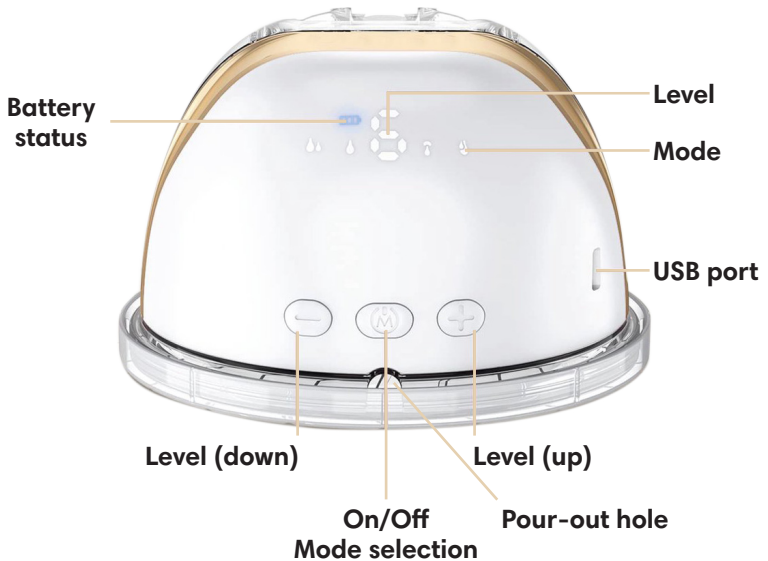Choose the right size breast shield reducer and insert it into the breast shield. Press the breast shield reducer firmly to prevent leakage.

Battery status:

Indicates the % battery remaining and charging status.

Low battery:

0-10% Battery turns red

Medium battery:

11-75% Battery turns white

Full battery:

76-100% Battery turns blue

When the battery needs charging the icon flashes red.

When the battery is charging, the icon flashes blue/white.

When the battery is fully charged the icon turns blue.

(Battery life depends on the modes and levels used)

Level + or -:

Use this to select the level used to pump. Depending on the selected mode, 1 - 6 levels are available. Each level up increases the suction power. (Always start at level 1 and build up to the desired level)

On/off & mode selection

Press and hold the button for 2 seconds to switch the flask on. Press again to select the mode. Press and hold the button for 2 seconds to turn the flask off.

USB port

Use the supplied charger to charge the flask. Insert and remove the charger carefully. Do not use excessive force, this may cause damage.

Mode

The display shows the current mode, each mode has a different function. Using the **M** button, you can change the mode.

Level

The display shows the current level; each level increases the suction power. Using the + and - button, the level can be changed.

Timer

The pump has a built-in timer. After 30 minutes of pumping, the pump automatically goes into 'sleep mode'. You can turn the pump back on after this.

Memory mode

The flask remembers the last selected level in the mode. If you switch between modes, the flask will continue at the last stopped level.

Massage/stimulation mode (1 - 5 levels)

It is advised to massage the breasts for 2 minutes before pumping. This improves the yield and convenience during pumping.

Expression/pump mode (1 - 6 levels)

Use expression/pump mode to pump fully. Each level offers different strengths while pumping.

2-in-1 mode (1 - 6 levels) 🍓

Can be very helpful for sensitive breasts and supports milk yield in a short time.

Dubbele frequentiemodus (1 - 4 levels) 🍓

Can be very helpful for shortened or retracted nipples.

Note: Build up the levels slowly in each mode. Start at level 1 and build up to the desired level.

Breast shell reducers

Choosing the right breast shield reducers is important for optimal results. It ensures a better fit and prevents leaks. The table below will help you make the right choice.

Nipple diameter for pumping (mm)	Nipple diameter after pumping (mm)	Suggested size (mm)
14-15-16	16-17-18	19mm
17-18-19	18-19-20	21mm
20-21-22	21-22-23	24mm
24-25-26	25-26-27	28mm

Inserting the pump

Bend forward slightly and centre the nipple in the flask. Make sure the pump is tight against the breast.

Once the pump is in position, stay upright with the upper body. Make sure the pump is sitting up and start pumping.

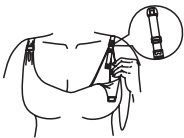
Once you have finished pumping, lean forward to release the pump. Bend forward slightly to avoid spilling milk.

Bra - strap

Adjust the length of the bra with the straps. This can make pumping more comfortable.

The end of the bra strap can be linked to the nursing bra.

The bra strap length can be adjusted by pulling the bra strap tighter.

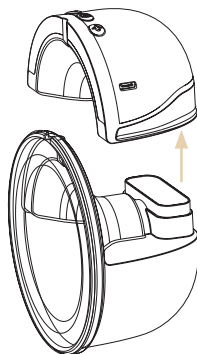
Pouring out

Pouring out the milk is easy via the pouring hole at the top of the collection tray. Always pour carefully without the motor sitting on the collection tray. This reduces the risk of moisture and milk damage.

You can refer to the instruction video for the correct way of pouring. The images below show step-by-step how to prepare the flask for optimal pouring.


Step 1: Remove the motor

the motor first. Do this carefully to avoid leaks. Leave the gas storage in place. Otherwise, there is a risk of a leakage.

Step 2: tilting and pouring

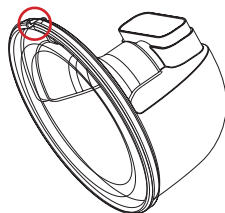Hold the breast pump by the breast shield, this is easiest to pour and gives control. As pouring improves, rotate the breast pump further and further until the receptacle is empty.

WARNINGS

- Use the breast pump only for expressing breast milk. Any other use is considered improper handling of the product.
- Consult a doctor or lactation consultant before using the breast pump, especially if you have medical questions or are experiencing breastfeeding problems.
- Check the breast pump and all parts for damage or wear before each use. Do not use damaged parts. If you find any defective parts, contact us immediately.
- Wash your hands thoroughly before assembling or using the breast pump.
- Clean the breast pump and accessories according to the instructions in the manual after each use to prevent infections.
- Only use the supplied charging cable. Using an incorrect power source may cause damage.
- Keep the motor and the adapter dry and away from water or other liquids.
- Store the breast pump and accessories out of the reach of children and pets.
- Stop pumping and consult a healthcare provider if you experience pain or discomfort while using the breast pump.
- Store breast milk in clean bottles or bags and at the correct temperature.
- Ensure the suction strength and speed are set correctly to prevent discomfort and possible damage to the nipples or breasts.

- Do not use the breast pump on irritated or damaged skin. Wait until the skin is healed before using the breast pump again.
- Do not use the breast pump while sleeping or resting. Stay alert and aware during pumping.
- Store breast milk in clean, well-sealed containers and label them with the date of expression.
- Never heat breast milk in a microwave, as this can cause uneven heating and destroy the nutrients in the milk. Use a bottle warmer or a warm water bath.
- Choose a quiet, comfortable place to pump to reduce stress and promote milk production.
- Pumping at regular intervals helps maintain a consistent milk supply.
- Vary the modes and settings to prevent habituation. This can improve yield.
- Do not put the parts in the dishwasher. This can damage the pump and is not covered under warranty.
- Never let the motor come into contact with water or moisture. This can affect the durability of your motor. Water or moisture damage is not covered under warranty.
- Keep your motor free from dust and milk residues. This can affect the durability of your motor.
- Do not drop your motor! Drop damage can cause serious internal damage and cause your pump to malfunction. Drop damage is not covered under warranty.
- Do not use chemicals, soaps, or other substances that may contain harmful ingredients. These can pose a danger to the product and the user.

- Read and follow the entire user manual carefully to use the breast pump correctly.
- If a defect occurs, contact Vulpes Goods immediately. Do not attempt to repair the pump yourself and stop using it immediately. A defect can be dangerous.
- If there is a reduction in suction strength, we recommend replacing the gas storage. This can help restore suction power.

FAQ

Q. My breast pump has low suction?

A. The gas storage and pump unit are not properly installed in place. When the pump is installed in the gas storage unit, it should be fully depressed.

A. The duckbill valve is not installed in place or is damaged. Check the part for correct placement and possible damage.

Q. Milk flows back.

A. If the gas storage is damaged or deformed, milk will flow back into the pump. In this case, stop using it immediately. Replace the gas storage to restore the connection.

A. The duckbill valve contains a crack. This gives the milk a chance to flow back. Check the valve for cracks, damage and proper placement.

Q. Pumping hurts/pumping does not work.

A. You may have a breast congestion or blocked milk duct. Ask a doctor to massage the breast.

A. Get the right breastplate size. Choosing the right size is important for a proper fit.

A. The pump level used is too strong. Start the pumping session from the lower level and build up slowly.

Q. Milk stays on the breastplate.

A. Check that you are using the right breast shield reducer. Sometimes a size that is too small or too large can cause the milk not to flow properly.

Q. My breast pump is not charging properly?

A. Make sure you use the USB cable provided. Connect it to a suitable adapter with the correct voltage.

CLEANING & MAINTENANCE

Non-washable parts

The main pump unit should never come into contact with moisture, water and/or milk residues. The pump must remain dry. A clean, dry cloth can be used to clean the main pump unit. Keep this area dry and free of residue. Any other cleaning method is covered by the warranty as abuse.

Washable parts

Breast shield, receptacle, silicone gas storage, silicone duckbill valve, breast shield reducers.

Cleaning washable parts that come into contact with milk prevents the build-up of dry milk residue, which can cause the growth of bacteria that can be harmful to the product, you and your baby.

After each use

1. Soak in mild detergent solution for 2 minutes and rinse under clean running water.
2. Clean immediately with drinking or purified water.
3. Let the parts air dry or pat dry with kitchen paper. This will not give off dust particles that are potentially harmful.

Do this once a day


1. Boil in water for five minutes. Make sure the water is deep enough to completely cover the parts. Watch the stirring accessories during cooking to avoid deformation due to uneven heat.
2. We do not recommend microwave or steam sterilisation. Keep in mind that some steam and microwave sterilisers reach high temperatures, which can melt plastic parts.
3. We do not recommend ultraviolet sterilisation.

PRODUCT PARAMETERS

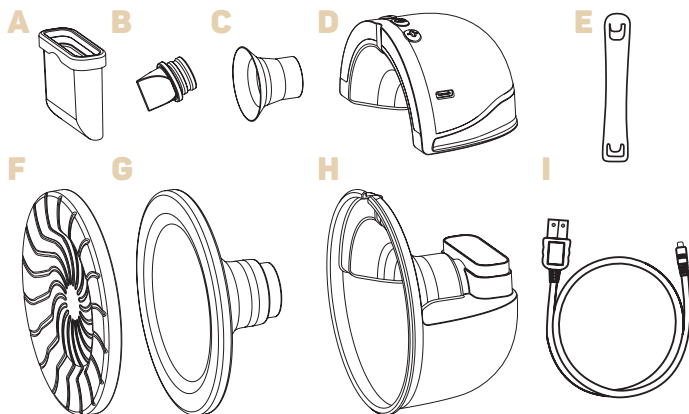Model:	VG-586
Weight (1 breast pump):	260Gr
Product size:	11.5cm x 7cm x 11.5cm (LxWxH)
Rated output:	5W
Input:	100-240V, 50/60Hz
Output power:	5V  1A
Suction strength:	< 0.0035 MPa
Battery capacity:	1500 mAh
Modes:	4
Collection tray capacity:	150ML

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